TOP FRONT

[TOP BACK](#)

blink-n-clean®

15ml

(EN) For clearer, more comfortable contact lenses during wear

blink-n-clean® drops are formulated to give contact lens wearers clearer, more comfortable lenses throughout the day without the need to remove them first. This gentle solution does more than just re-wet; it washes away natural debris and protein build up to leave lenses clear, fresh and comfortable. Particularly useful for extended wear lenses or when wearing lenses for longer periods of time.

NO NEED TO REMOVE LENSES

Directions for use:

- Apply one or two drops to each eye up to four times daily
- Blink to wash the lens

Precautions:

All instructions must be followed. Failure to do so may result in serious eye infection or injury.

- Do not use if tamper-evident seal is broken or missing
- Close the bottle tightly immediately after use
- Do not use if allergic to any of the ingredients
- Do not use more than four times a day
- Do not drink the solution
- Store between 5°C and 25°C
- Keep out of reach of children
- Do not use after the expiry date
- Discard any remaining solution 60 days after first opening the bottle
- To avoid contamination do not touch bottle tip on any surface

Contains:

- Sodium Chloride
- Tromethamine
- Hydroxypropylmethylcellulose
- Tyloxapol
- EDTA
- Polyhexamethylene biguanide 0.0001%
- Purified water

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Do not use if package is damaged
Ne pas utiliser si l'emballage est endommagé

AMO Ireland, Block B, Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland

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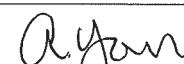
blink-n-clean is a trademark owned by or licensed to Abbott Laboratories,
its subsidiaries or affiliates.

© 2015 Abbott Medical Optics Inc.

AM70434UO10A
Revision Date: 04/2015

10ml INSERT (AUTO)
207mm x 117mm
AM01194

**ENTIRE INSERT TO PRINT OUT OF PROCESS BLACK
DO NOT PRINT CUTTER GUIDE OR CALL OUTS**

ARTWORK SPECIFICATIONS		
Part No: AM70434UQ10A		
Platform: Mac Illustrator CS2		
	SIGNATURE	DATE
Packaging Coordinator		6/1/15
Marketing		5/15/15
Regulatory		4/29/15
Updated: 07.05.15 / 12.00		
Version No: 2		

Swatches used:

			Black

FOR AMO INTERNAL USE ONLY

Date:

☐ Component No. ☐ Drawing No. ☐ Drawing Version No. ☐ Barcode ☐ Address ☐ Sterile A ☐ Recycle Symbol ☐ Hour glass (inserts) ☐ Language lozenge	☐ CE Mark (size, location) ☐ Lot and Expiry (P or P) ☐ Callouts ☐ Swatches ☐ Cutter not obscuring copy ☐ Number of blacks ☐ No four-colour splits ☐ Artwork prepared for print ☐ Project specific
---	--

Revised checklist Feb 2004.